



3D/MIP/MPR User Guide for Synapse Mobility

In order to view 3D reconstructions, maximum intensity projections and multi-planar reconstructions for CTs on Synapse Mobility, click on the series you would like to view. Please wait until the images have loaded completely before doing this.

- 1) Click to open the View Menu.

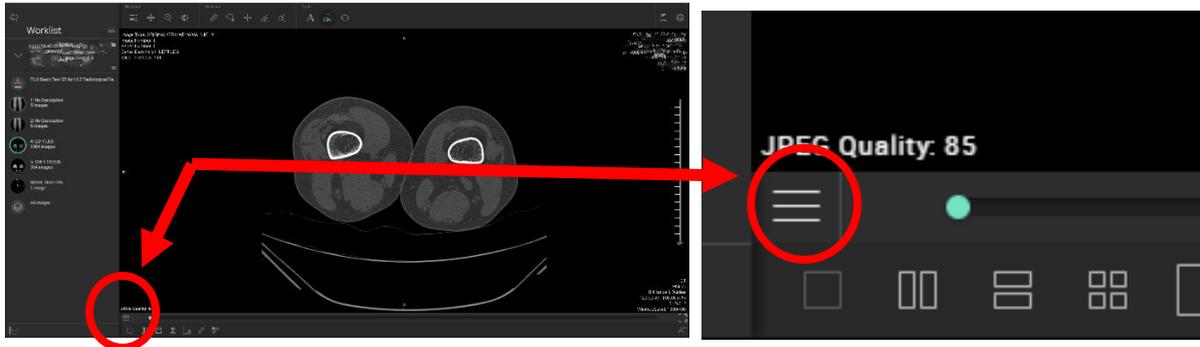


Internet browser

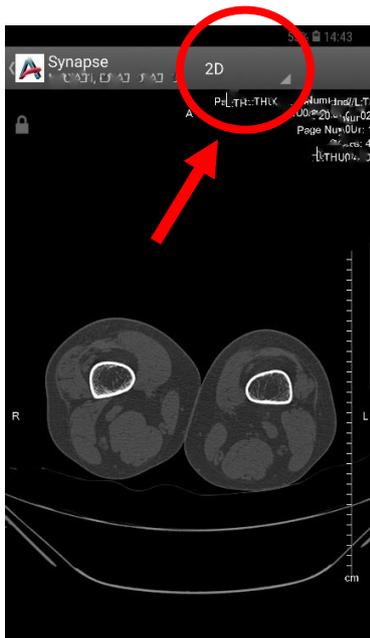


Mobile application


2) Click on 3D or MIP/MPR.

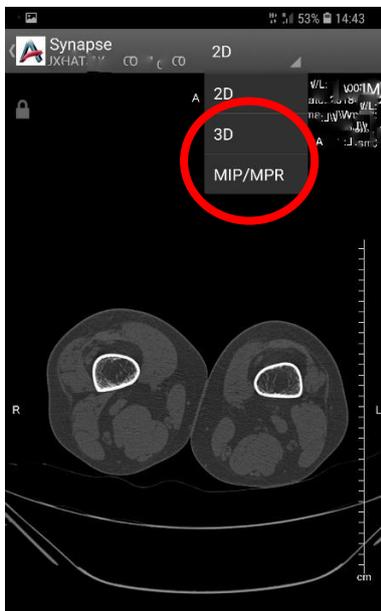

3) For MPR in internet browser: Double click on the view that you would like on the left to maximise image.

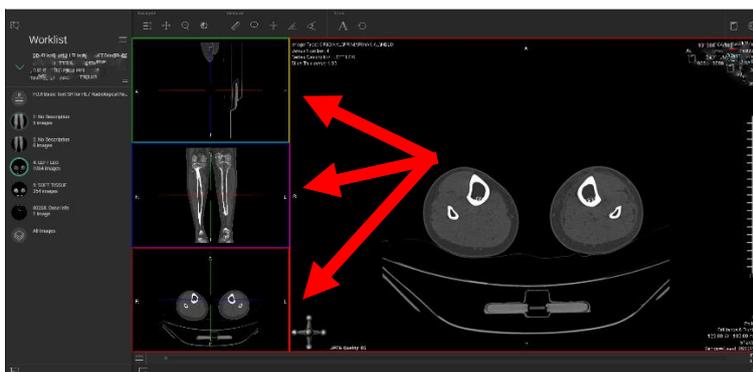

For MPR in mobile app: Swipe left or right to toggle between views.

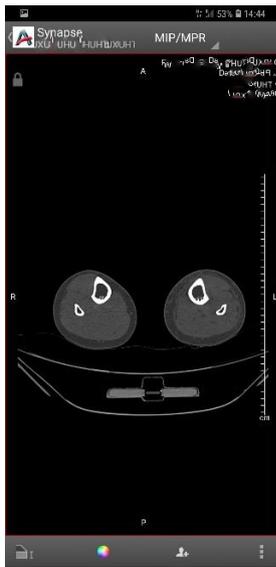

Axial



Sagittal



Coronal